



**LIGHT LIVESTOCK EQUIPMENT**  
BY A USABLE VALLEY ALPACAS  
LIGHTLIVESTOCKEQUIPMENT.COM  
(866) 999-AVA1(2821)

## Alpaca & Llama Rope Restraint Instructions

Thank you for your purchase of TOP GUN ALPACA / LLAMA RESTRAINT ROPES. Please read the following instructions to ensure safety for animals and handlers.

Select two secure anchor points 16 to 18 feet apart about 3 inches above ground level. Tie or clip ropes without pulleys to one anchor point and clip carabiner on pulley end to the other. Lay ropes out prior to bringing animal.

Three people are needed to restrain / release the Alpaca: One person at the head of the Alpaca, one at the tail, and one to work the ropes.

Bring Alpaca to area where ropes are laid out, head at pulley end of ropes. Persons at head and tail of Alpaca steady the animal while third person slips looped end of ropes on each leg above the pastern joint. When all 4 loops have been put on legs, person handling rope grasps loose end of rope by the knot at pulley end and swiftly moves towards rear of animal while two people holding Alpaca guide it to the ground.

Swift, steady pull by the rope handler is essential for smooth take-down. Rope handler should continue to pull until Alpaca is fully stretched and restrained.

Once Alpaca is down, it should not be left unattended.

When releasing the Alpaca, pull backwards slightly on knotted end of rope to snap it out of the cam cleats. Then Alpaca can be pulled rear-ward slightly to provide slack to remove loops from legs. Two persons can apply some downward pressure to neck and hips while third person removes loops.

